- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

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## Stethoskope

QM-1-025J

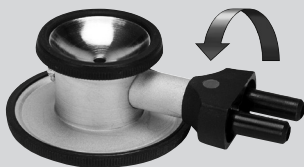
<b>de</b>	Stethoskope .....	6
<b>en</b>	Stethoscopes .....	8
<b>fr</b>	Stéthoscopes .....	10
<b>it</b>	Stetoscopi .....	12
<b>es</b>	Estetoscopios .....	14
<b>pt</b>	Estetoscópios .....	16
<b>ru</b>	Стетоскопы .....	18

- de** z.B. Noroscop umschaltbar
- en** For example: Noroscop dual-function
- fr** p.ex. Noroscop commutable
- it** ad es. Noroscop con canale selezionabile
- es** p.ej. estetoscopio Noroscop con válvula selectora
- pt** por ex. Noroscop comutável
- ru** например: Noroscop с переключаемой головкой

**Rapport**

**Kirchner Colorsop  
Plano**

**Kirchner Colorsop  
Duo**

**Single**

**Double**

**Prestige-Baby**

**Prestige-Baby  
light**

**Prestige-Child**

**Prestige-Child  
light**

**Prestige-Standard**

**Prestige-Standard  
light**

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Dear Customer, thank you for choosing a KaWe product. Our products are known for their high quality and longevity. This KaWe product fulfils EC Standards 93/42/EEC (standards for medical products).

**Please read these instructions thoroughly and carefully before attempting to use this product and heed the care instructions.**

Familiarise yourself fully with this product before using it.

**Use:** The stethoscopes are only to be used by authorised personnel. The stethoscope consists of binaural pieces, tubes and a breast piece. The stethoscope breast piece is to be placed lightly on the desired area of the patient's body according to the type of examination being performed. Double-sided stethoscopes allow users the option of selecting between two different input channels (funnel or membrane).

**Purpose:** The stethoscope is a diagnostic instrument used for auscultation.

**Operation:** The basic principle behind the use of a stethoscope is the "oscillating air column". A closed air column is required for sound to be transmitted. When putting on the binaural stethoscope, adjust the binaural pieces in such a way that the olive-shaped ear tips are pointing slightly forward — toward the patient's face — so that the instrument fits to the construction of the ear. Place the membrane and/or the funnel on the area of the body that is to be examined. The following models

are equipped with a dual-function membrane/funnel: Planet, Top Cardiology, Standard-Prestige, Child-Prestige, Baby-Prestige, Noroscop, Colorscoop Duo, Duoscoop, Double und Baby-Colour. To select between the funnel and membrane functions, the user need only to twist the head of the stethoscope. In each case, depending on the selected function, one air column is closed and the other is opened.

**Unsuitable use/contraindication of the product:** Any use other than that described here is not in accordance with the intended use of the instrument. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

**Further information, maintenance, storage:** When used and stored properly, this stethoscope will serve you many years.

**Guarantee:** When used under normal circumstances and with attention to these instructions (the User's Manual) we guarantee this product for two years after the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer.

**Warning instructions:** Never let the stethoscope come into contact with water.

**Limitation of reuse preparation:** Frequent reuse only minimally affects the instruments. The working life of the instruments usually depends on the wear and tear and damage by use.

## Instructions

**Workstation:** Remove surface impurities with a disposable/paper towel.

**Storage and transport:** In order to prevent contamination of the instrument or contact with dust, always store the stethoscope in its case.

**Cleaning preparation:** No special requirements.

**Manual cleaning:** The head piece and plastic hoses can be wiped off with alcohol or a mild soap solution. The ear tips are to be unscrewed and can be washed with soapy water. The membrane can be washed with soap after the retaining ring has been removed.

**Automatic cleaning:** Automatic cleaning is not possible.

**Disinfection:** Disinfecting solutions and cleaning agents such as Meliseptol®/Meliseptol® rapid are to be used in accordance with the manufacturer instructions. Please heed the given reaction times! Never use abrasive cleaning agents!

**Maintenance:** Defective parts must be removed and/or replaced.

**Check and performance test:** Check the membrane, hose and head piece for damage. The instrument must be complete. Before use, the ear tips must be positioned correctly. The position of the head piece is to be adjusted by each user individually.

**Packaging:** Individual packaging: a standard polyethylene bag can be used. The size of the bag must

correspond to the size of the instrument so that it closes properly.

**Sterilization:** As the stethoscope is not intended for use during operational procedures, simple cleaning and or disinfection with an alcohol-based surface disinfecting agent is sufficient.

**Storage:** Store the stethoscope in a dry place.

**Additional information:** Please always heed these instructions.

**Common accessories:** Further information about this product can be found at our internet shop at: [www.kawemed.de](http://www.kawemed.de).

Manufacturer: **KaWe**

**Contact with the manufacturer:** Address or telephone number of the dealer or dial +49-7141-68188-0.

**Disposal:** Dispose along with normal domestic waste.

## Symbol key:

	Manufacturer
	Date of manufacture
	Heed the User's Manual
	Warning!
	Complies with relevant EU-guidelines

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**KIRCHNER & WILHELM** GmbH + Co. KG