
- D** **F.O. Kopfleuchte H-600**
Gebrauchsanweisung
- GB** **F.O. Head lamp H-600**
Users Manual
- I** **F.O. Caschetto H-600**
Istruzioni per l'uso
- F** **F.O. Eclairage frontal H-600**
Mode d'emploi
- E** **Lámpara frontal de fibra óptica H-600**
Instrucciones de empleo
- P** **Lanterna de cabeça de fibras ópticas H-600**
Manual de operação
- RUS** **Налобный осветитель H-600**
Руководство по применению

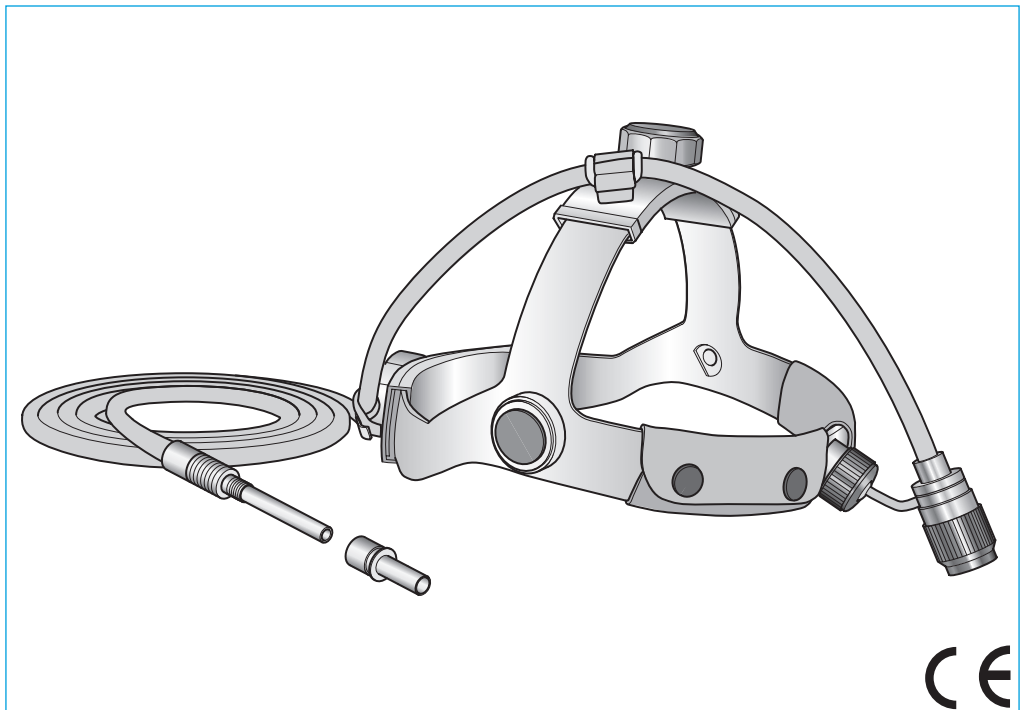QM - 1 - 163B M09

2008/06

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GB Users Manual

Purpose

- The light-intensive head lamp is destined for use in the operative and in the otorhinolaryngologic area.
- The head lamp is only to be used with halogen light sources of up to max. 150 W.
- The head lamp may not be connected to a xenon light source.

Guarantee

When used under normal circumstances and with attention to these instructions (the users manual) we guarantee this product for two years after the date of purchase.

Should you need further information or should your instrument require repair, please contact your dealer.

Safety instructions!

Use

- *The head lamps are only to be used by authorized personnel.*
- *The instructions given by the manufacturer of the light source must be followed.*
- *The head lamp may not be connected to a xenon light source.*
- *The head lamp is only to be used when it is in good working order and condition.*
- *The light conducting cable may not be removed with a snapping motion! Smallest bending radius 50 mm.*
- Intense snapping motion may result in breakage of the fibre optics.
- *Do not suspend the head lamp from the light conducting cable or carry it by means of the light conducting cable.*
- *Protect the head lamp against heat, oil and caustic solvents.*

Cleaning

- *Ultrasonic cleaning is not allowed!*
- *The head lamp may not be brought into contact with water or solutions.*
- *Sterilization in hot air devices or in autoclaves is not allowed!*

CE Marking

This KaWe product meets the standards for medical products 93/42EEC.

Technical Data

Ergonomic headband

- Continuously variable height and width adjustment
- The headband padding is removable.

Light conducting cable

- Length: 2300 mm
- Fiber bunch: 3.5 mm
- The light conducting cable is electrically insulated.

Light exit

- The light spot is infinitely focusable.

Total weight

(with cable) approx. 450 g

Accessory: adapter for light sources

(not included in the scope of supply)

- Adapters that can be screwed off are available for the light sources of all common manufacturers (see page 16).
- More information about KaWe products can be attained in www.kawemed.de

Cleaning

- Wipe off the head lamp with a damp cloth and with surface disinfectants immediately after each use.

Pre-disinfection

- Pre-disinfection is to be carried out in accordance with the instructions given by the manufacturers.

Disinfection

- Disinfectants and cleaning solutions must be prepared and proportioned in accordance with the instructions of the manufacturer. The prescribed concentration and reaction time must be followed.
- Possible disinfectant residues at the light exit and the optics must be removed carefully.

Maintenance


- When used and cleaned appropriately, the head lamp remains maintenance-free.

Storage

- Carefully remove the light conducting cable from the light source.
- Store the head lamp in a dry place.
- Do not suspend the head lamp from the light conducting cable.


A Tighten the light conducting cable

1. Insert the cable
2. Close the fixing strap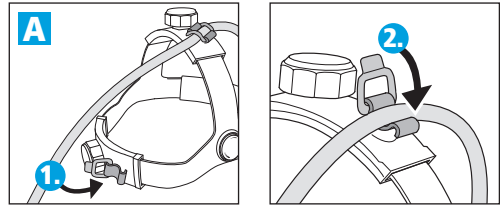

B Adjust headband

1. Adjust width of head and
2. height of headband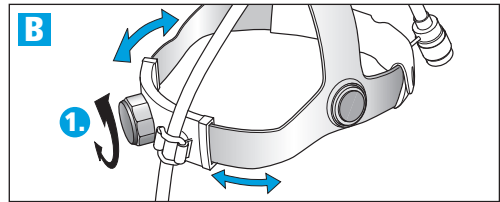


C Halogen light source

- Push the plug with the suitable adapter into the halogen light source.
- Turn ON
- Regulate the lightness by means of the light source.
- Carry out inspection and operational check prior to every start-up!


! Attention!

Prior to periods of non-use remove the plug of the light conducting cable from the light source.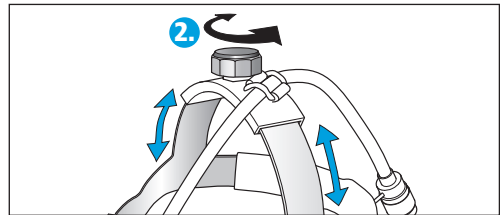

D Adjust direction of light


1. Unscrew the knurled head screw.
2. Adjust desired light direction.
3. Tighten the knurled head screw.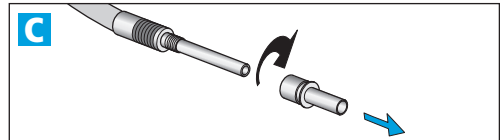

E Adjust light spot



1. Infinitely adjust large or
2. small cone of light

Distance	20 cm	30 cm	50 cm
Light spot ø	55 - 130 mm	85 - 200 mm	140 - 350 mm

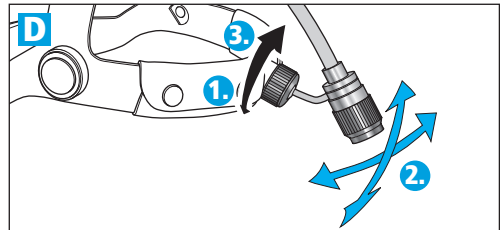F Remove headband padding

1. Open the snap fasteners, remove headband padding

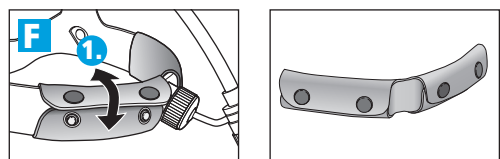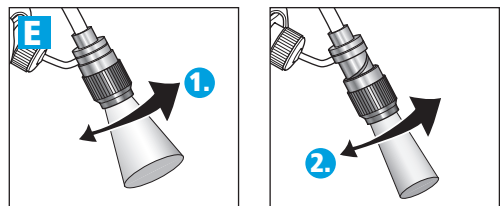Subject to technical modifications without prior notice.

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KIRCHNER & WILHELM GmbH + Co. KG